

Natrelle® Breast Implants IMPORTANT SAFETY INFORMATION AND APPROVED USES

Breast implants are not considered lifetime devices. The longer people have them, the greater the chances are that they will develop complications, some of which will require more surgery.

Breast implants have been associated with the development of a cancer of the immune system called breast implant-associated anaplastic large cell lymphoma (BIA-ALCL). This cancer occurs more commonly in patients with textured breast implants than smooth implants, although rates are not well defined. Some patients have died from BIA-ALCL.

Patients receiving breast implants have reported a variety of systemic symptoms, such as joint pain, muscle aches, confusion, chronic fatique, autoimmune diseases, and others, Individual patient risk for developing these symptoms has not been well established. Some patients report complete resolution of symptoms when the implants are removed without replacement.

Please see additional Important Safety Information on the following pages.

# EVERYONE has a story













#### Natrelle® Breast Implants IMPORTANT SAFETY INFORMATION (continued)

#### Who can get breast implants?

Natrelle® Breast Implants are approved for the following:

- Breast augmentation for women at least 22 years old for silicone-filled implants and for women at least 18 years old for saline-filled implants. Breast augmentation includes primary breast augmentation to increase the breast size and revision surgery to correct or improve the result of a primary breast augmentation
- Breast reconstruction. This includes primary breast reconstruction to replace breast tissue that
  has been removed due to cancer or trauma or that has failed to develop properly due to a severe
  breast abnormality. This also includes revision surgery to correct or improve the result of a primary
  breast reconstruction

#### Who should NOT get breast implants?

Breast implant surgery should NOT be performed in:

- Women with active infection anywhere in their body
- Women with existing cancer or precancer of their breast who have not received adequate treatment for those conditions
- Women who are currently pregnant or nursing

**GABBY** had a specific goal in mind. She wanted to enhance her curves while maintaining her shape. With *Natrelle®*, she added subtle volume that met her unique needs.

**JENN** received a revision with *Natrelle*\* to give her the fullness she always wanted. Feeling confident in her decision, she was able to advocate for exactly what she wanted and is now in love with her new look.

**KATHY** wanted breast augmentation for a long time but was always hesitant. She never stopped wanting her dream look, and at 50, she was finally ready. She didn't want anything to stand in her way anymore of choosing herself—and she is so glad she did.

#### Natrelle® IMPORTANT SAFETY INFORMATION (continued) What should I tell my doctor?

Tell your doctor if you have any of the following conditions, as the risks of breast implant surgery may be higher:

- Autoimmune diseases (eg, lupus and scleroderma)
- A weakened immune system (eg, taking medications to decrease the body's immune response)
- · Planned chemotherapy or radiation therapy following breast implant placement
- . Conditions or medications that interfere with wound healing and blood clotting
- · Reduced blood supply to breast tissue
- Clinical diagnosis of depression or other mental health disorders, including body dysmorphic disorder and eating disorders
- Those with a diagnosis of depression or other mental health disorders should wait for resolution or stabilization of these conditions prior to undergoing breast implantation surgery



Actual *Natrelle* patients. Individual results may vary.





#### Natrelle® IMPORTANT SAFETY INFORMATION (continued) What else should I consider?

- There is a Boxed Warning for breast implants. Please see bold text at beginning
- Many changes to your breasts following implantation are irreversible. If you later choose to have your implants removed and not replaced, you may experience dimpling, puckering, wrinkling, or other cosmetic changes, which may be permanent
- Breast implantation is likely not a one-time surgery. The longer implants are in place, the greater
  the potential risk for complications. You will likely need additional surgeries on your breasts due to
  complications or unacceptable cosmetic results. Thus, you should also consider the complication rates
  for later (revision) surgery since you may experience these risks in the future
- . Cancer treatments and surgery will affect the outcome and timing of breast reconstruction
- Breast implants may affect your ability to breastfeed, either by reducing or eliminating milk production



### You're not a carbon copy of someone else, so why should your look be? You deserve a customized look that feels uniquely you.

Whether you are looking to add fullness or regain volume, *Natrelle*\* offers the largest range of implant options to give a customized look for every curve, for every shape, for every BODY.

Natrelle®'s commitment to you is backed by 35 years of excellence and innovation in cosmetic and reconstructive surgery with the highest manufacturing and safety standards.

- Rupture of a silicone-filled breast implant is most often silent. Even if you have no symptoms, you should have your first ultrasound or MRI at 5 to 6 years after your initial implant surgery and then every 2 to 3 years thereafter regardless of whether your implants are for augmentation or reconstruction. If you have symptoms of or uncertain ultrasound results for breast implant rupture, an MRI is recommended. Additional imaging may be required depending on your medical history and status. The health consequences of a ruptured silicone gel-filled breast implant have not been fully established
- Routine screening mammography for breast cancer will be more difficult, and implants may rupture
  during the procedure. Perform self-examination every month for cancer screening and ask your
  surgeon to help you distinguish the implant from your breast tissue. Lumps, persistent pain, swelling,
  hardening, or changes in implant shape should be reported to your surgeon and possibly evaluated
  with imaging

# Your Breast Augmentation YOUR WAY







Full look with slight roundness on top

**Customizing** means choosing a look that fits you. Whether you desire dramatic cleavage or a subtle look, *Natrelle*® **offers options** to help you find a silhouette you'll love.

Based on your available tissue, body structure, and aesthetic goals, your board-certified plastic surgeon will help you choose the profile and size for a result that's right for you.



Fuller look with more roundness on top



Fullest look with a very rounded top

of primary breast augmentation patients

were satisfied with their *NATRELLE* implants 10 years after surgery.\*

<sup>\*</sup>Patient satisfaction based on 279 primary augmentation patients using round implants in the Allergan\* core study (86% definitely satisfied and 8% somewhat satisfied).

# A Customized Look JUST FOR YOU



LOW 110 cc - 610 cc



LOW PLUS 125 cc - 640 cc



MODERATE 140 cc - 755 cc

#### Natrelle® IMPORTANT SAFETY INFORMATION (continued) What are key complications with breast implants?

Key complications include reoperation, implant removal with or without replacement, implant rupture with silicone-filled implants, implant deflation with saline-filled implants, and capsular contracture (severe scar tissue around the implant). Other complications include breast pain, swelling, asymmetry, wrinkling/rippling, implant malposition nipple complications, hypertrophic scarring, and implant palpability/visibility.

Talk to your doctor about other complications.

Finding your customized look means having options. And who doesn't like options? *Natrelle®* ensures everybody receives a customized look by being the ONLY breast implant brand to offer 5 implant projections and 3 gummy gel levels. With these options, the *Natrelle* INSPIRA® collection has a total of 300 implants—the most in the market—to choose from.

You deserve a look customized to you.

#### **5 Projection Options**

#### **ENDLESS POSSIBILITIES**



For more information, see the patient brochures at www.allergan.com/products.

To report a problem with Natrelle® Breast Implants, please call Allergan® at 1-800-624-4261.

The sale and distribution of Natrelle® Breast Implants is restricted to licensed physicians who provide information to patients about the risks and benefits of breast implant surgery.



The #I surgeon-selected and the most requested implant brand in the US.†



*Natrelle*<sup>®</sup> is dedicated to providing you with quality, consistency, and transparency. The *Natrelle*<sup>®</sup> **Breast Implant Collection** includes:

- Cohesive silicone gel that stays in place
- Higher gel fill ratio
- Smooth-surface options, always
- Thick protective shell

#### YOUR SAFETY OUR PRIORITY

Each *Natrelle*<sup>®</sup> implant is made by hand and goes through an extensive period of rigorous testing and quality checking to ensure your safety.



State-of-the-art facility designed specifically to produce breast implants



Each implant goes through an intensive 12-day production



Multiple quality checks through the manufacturing process

#### A STRONG SHELL

#### For Your Gel

A look and feel that's just like you! *Natrelle* INSPIRA® Breast Implants are engineered with a cohesive gel that holds together and a robust shell that's designed to keep your gel in place. The aesthetic benefit? Our shell + gel together help maintain shape, create upper pole fullness, and retain a soft feel.

\*Significance inside the body has not been established.

#### **Upper Pole Fullness**

This term refers to the shape and contour your breast implants add to the top portion of your breasts above the nipple. The more volume added, the more increased cleavage.



#### **SOFT**

#### To the Touch

The Natrelle INSPIRA®
Collection offers three
levels of gummy to ensure
they feel just like you and
create your desired shape.



### How gummy do you want to go?

It's not just about the look, it's about the feel. Gummy refers to the cohesivity, or consistency, of the silicone gel in *Natrelle* INSPIRA® Breast Implants. Whatever look and feel you're trying to achieve, *Natrelle* INSPIRA® is the only implant brand that offers three gummy gel levels ranging from soft, softer, and softest.







# The Natrelle ConfidencePlus WARRANTY PROGRAM Coverage for your Breast Implants

#### You're Covered!

When you choose *Natrelle* INSPIRA® Breast Implants, you're automatically enrolled in the *Natrelle® ConfidencePlus®* warranty program, so you can have peace of mind knowing you have comprehensive implant coverage.





#### **GEL WARRANTY**

FREE coverage and automatic enrollment

#### Rupture

- Implant replacement: Lifetime for both affected and contralateral implants
- Out-of-pocket financial assistance: Up to \$3,500 for 10 years

#### Capsular contracture Baker Grade III/IV

- Implant replacement: 10 years for both affected and contralateral implants
- Out-of-pocket financial assistance:
   Up to \$2,000 for 2 years
   (augmentation patients only)<sup>1</sup>

Scan here to learn more about our warranty program.



# The AESTHETIC of YOU

Actual *Natrelle* patients. Individual results may vary.



## FULLEST LOOK with a very rounded top

Primary Augmentation With Natrelle INSPIRA® Cohesive










Photos provided by Dr Allen Gabriel. **Individual results may vary.** 





# FULLER LOOK with more roundness on top

Primary Augmentation With Natrelle INSPIRA® Soft Touch

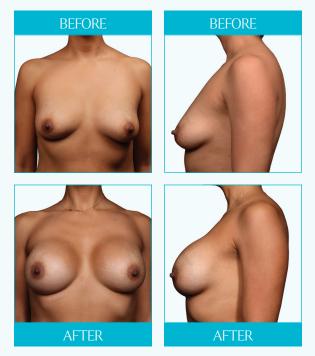

Photos provided by Dr Allen Gabriel. **Individual results may vary.** 



#### Primary Augmentation With Natrelle INSPIRA® SoftTouch



Photos provided by Dr Allen Gabriel. **Individual results may vary.** 



### VERY LITTLE fullness on top

#### Primary Augmentation With Natrelle INSPIRA® Soft Touch



Photos provided by Dr Allen Gabriel. **Individual results may vary.** 



#### Your New Look Comes With REWARDS


Natrelle PERKS\* is an exclusive offer for Alla Members who receive breast augmentation with Natrelle INSPIRA\* gel breast implants.

Allē members get 500 points to redeem for future Allergan Aesthetics treatments. In addition, with *Natrelle* PERKS®, patients may be eligible to receive a complimentary aesthetic treatment offer.



Scan here to learn more about *Natrelle* PERKS®.

Actual *Natrelle* patients. Individual results may vary.

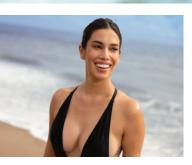









#### QUESTIONS to ask your plastic surgeon

Which *Natrelle* INSPIRA® Breast Implants are right for me?

How much upper volume can I achieve?

Can I see before-and-afters of patients who had similar aesthetic desires?

What if my desired aesthetic changes over time? What are my options?

Will implants affect the appearance of my nipples?

Will implants alone give me the look I want, or will I need an additional procedure?

How do I prepare for breast augmentation surgery?

Should I have the implants placed under or over the muscle?

Will my incision be in my breast crease, nipple, or armpit?

What are the possible complications with breast augmentation surgery?

What are the possible long-term complications of breast implants?

Will I need to have surgery again after my initial breast augmentation?

What is the lifespan of my implants?

Is there warranty coverage for my implants?

How is the recovery after breast augmentation surgery?

Will I need to change my lifestyle after breast augmentation?



Can I breastfeed after having implants?

Will my implants affect my ability to become pregnant?

How soon after surgery can I exercise?

Will the look of my augmented breasts change if I lose or gain weight?

How will my implants change with time as I get older?

If I choose to have my breast implants removed, how will my breasts look after?

What is maintenance like for breast implants? How often do I need check-ups?

How do I do my monthly breast exam with implants?



"IT WAS TIME TO BE WHO I WANTED TO BE AND LOOK THE WAY I WANTED TO LOOK."













"I FINALLY DID SOMETHING FOR ME."

#### Low Monthly Payment Plans

# for YOUR Breast Augmentation with PatientFi

#### Flexible Plans to Fit Your Budget

Create a customized payment plan that works for you.

#### Select Offers Ranging from 6 to 60 Months

Take advantage of longer terms for worry-free, low monthly payments!

#### No Hard Credit Checks—Ever

Everybody deserves a protected credit score.



#### Patient Fí.

#### The Friendly Way To Pay

#### Lower Patient Annual Percentage Rates (APRs)<sup>s</sup>

Patients financing their *Natrelle®* breast augmentation will automatically receive up to a 1% discount on their plan's APR.



Plans Available
Pay over time
with transparent,
zero interest

zero interest promotional plans. Plus, no hidden fees!

Apply for an Instant Decision

It only takes a few seconds and won't impact your credit score!

Important terms and conditions: All financing subject to credit approval. Depending on creditworthiness and choice of financing option, APBs range between 6.99% and 29.99%. Scheduled payments are based on full contract term length. Not every provider offers all product term lengths and promotions. Please inquire which term lengths and promotions are available.

Allergan Aesthetics is not a financing company and is not responsible for and does not support the financing referenced herein. All financing and support are provided by PatientFi. Patients and healthcare providers should contact PatientFi with any questions. Patients should apply directly to PatientFi for financing.







@natrellebreastaugmentation natrelle.com



